iNAP<sup>®</sup> One  
Sleep Therapy System

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
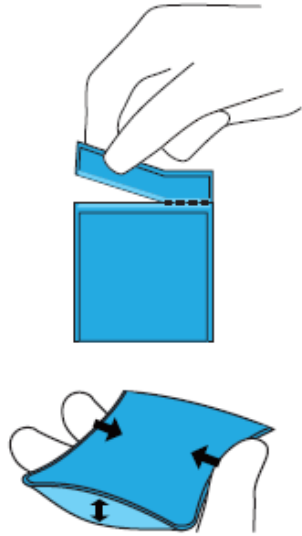
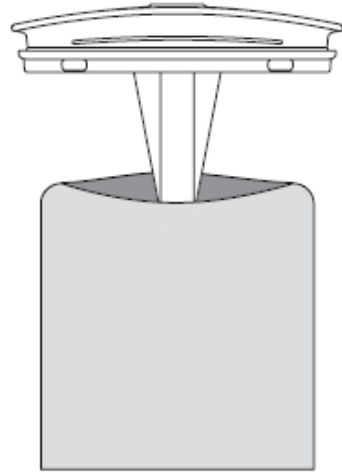
Tear the Dotted line of DryPad and squeeze to open it


Squeeze the side of DryPad

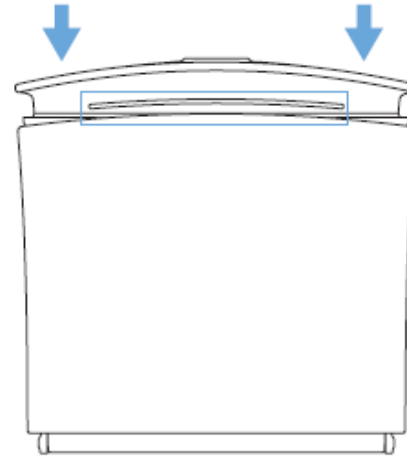
Make the position of marking line under the container body

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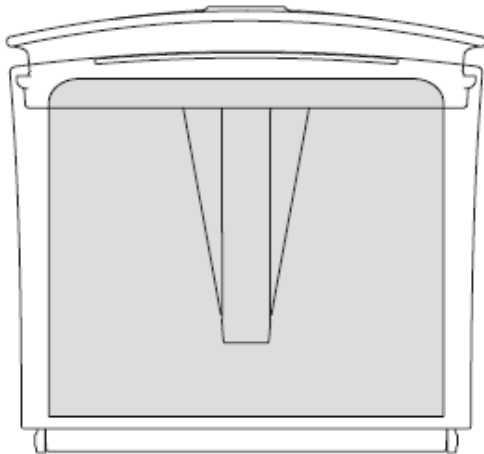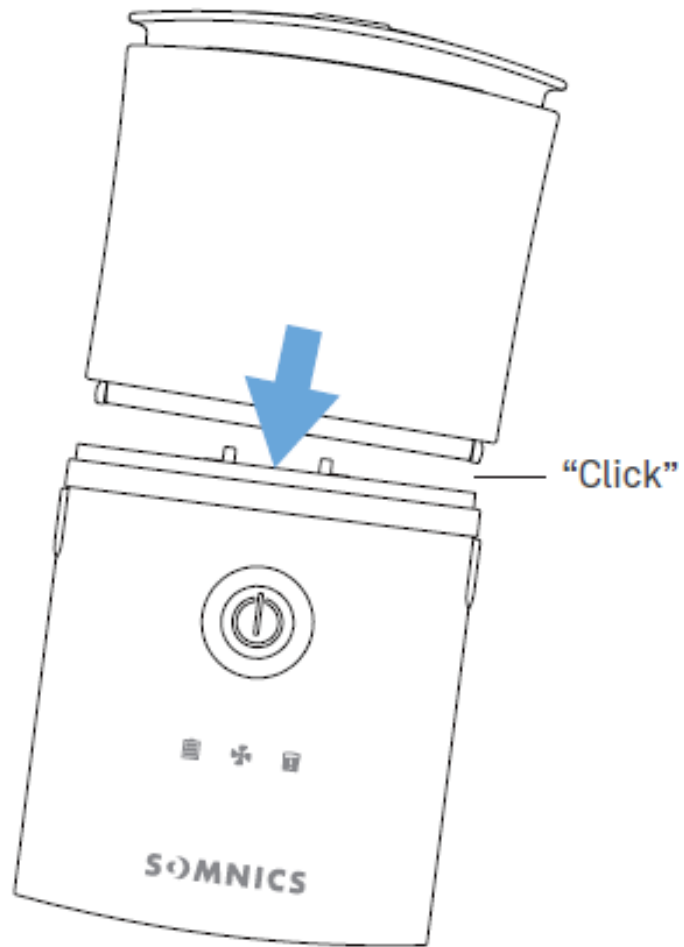
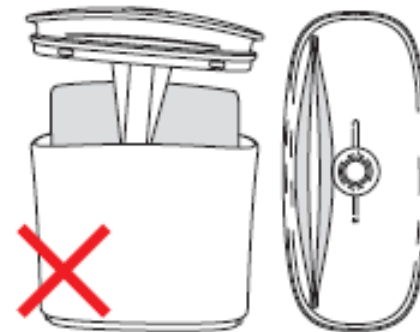
⚠ **Caution** Incorrect container cover position


⚠ **Caution** Incorrect DryPad position

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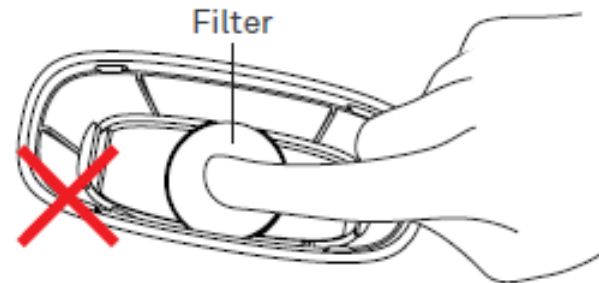
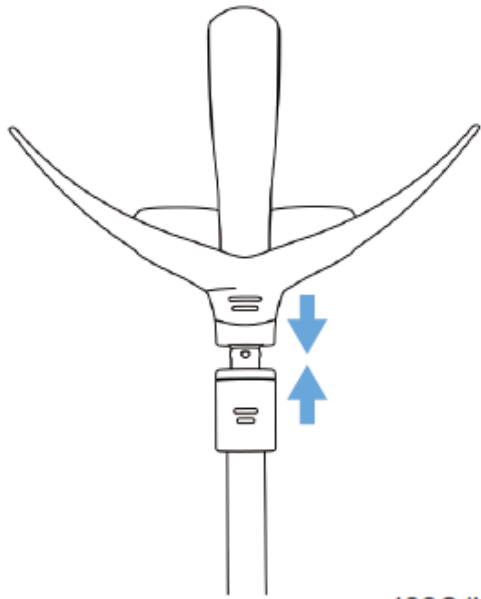
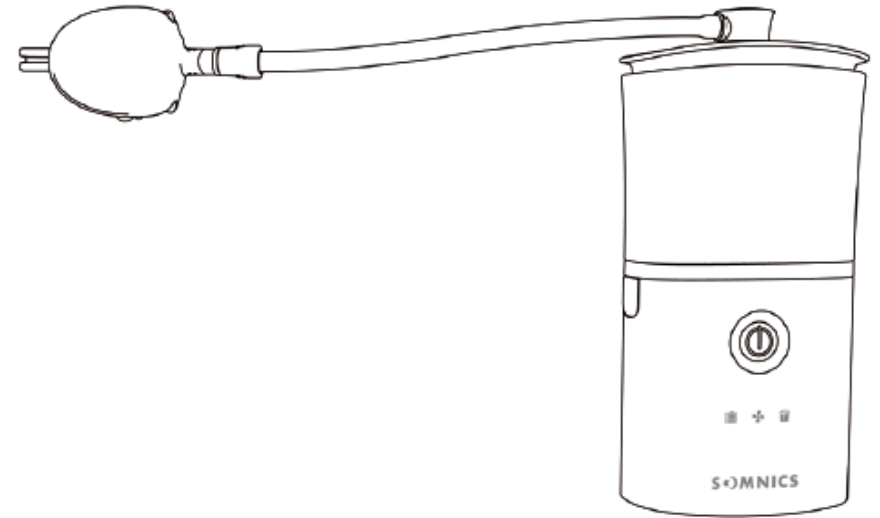
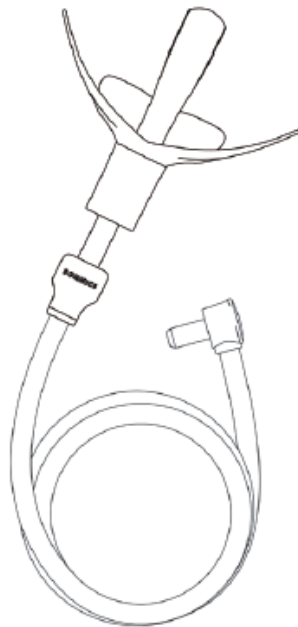
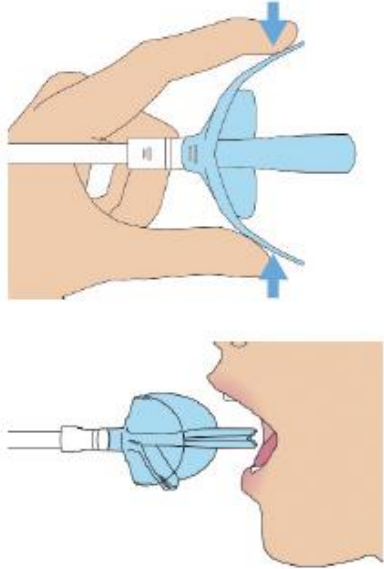
**Note** Wash the oral interface before using it at the first time.

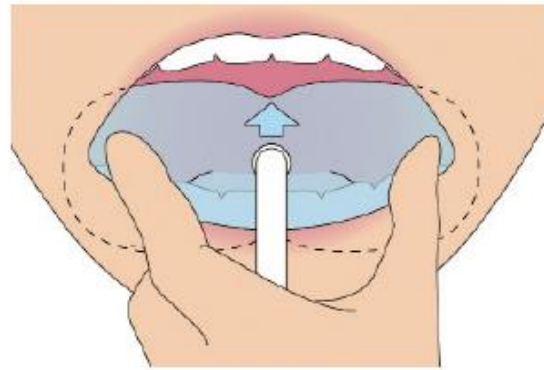Insert the oral interface in your mouth by squeezing the lip shield appropriately.

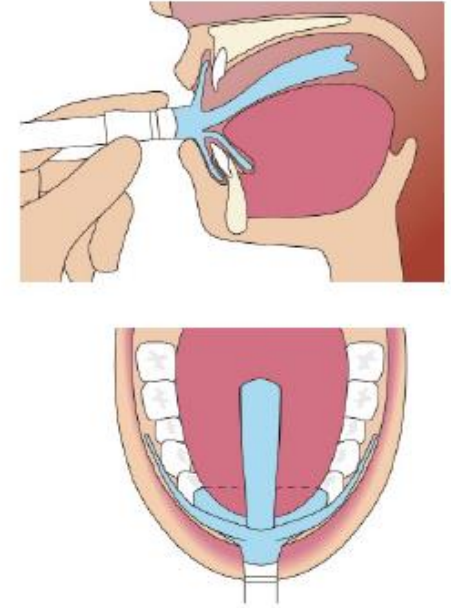Make sure the lip shield is located between the frontier of the teeth and the rear of the lips.


Keep the lip shield and the flexible tube in your mouth, then have the tongue rest on the tongue shield.

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
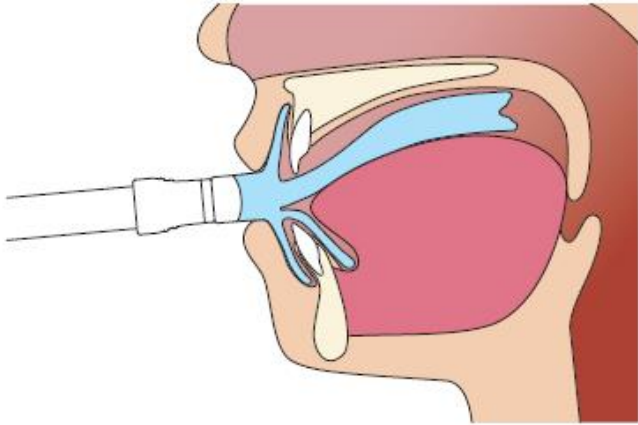
Close your mouth and get ready to start the treatment of the iNAP One Sleep Therapy System.

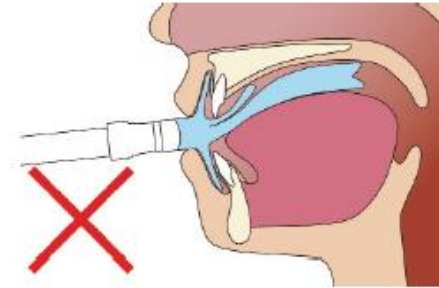- Note** Please put the oral interface ONLY in your mouth.
- Note** DO NOT chew on or swallow the oral interface.
- Note** DO NOT wear the oral interface while eating or drinking.

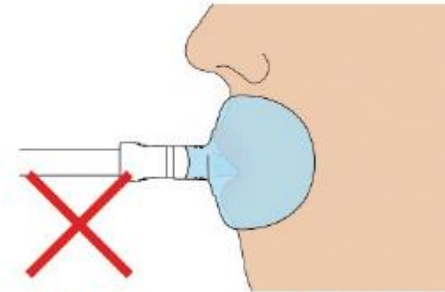
**⚠ Caution** DO NOT put tongue shield on the tongue inside your mouth.


**⚠ Caution** DO NOT put lip shield outside the lip. What's the tongue shield? Please refer to section 1.4.3


## Wearing the Oral Interface (IO8)

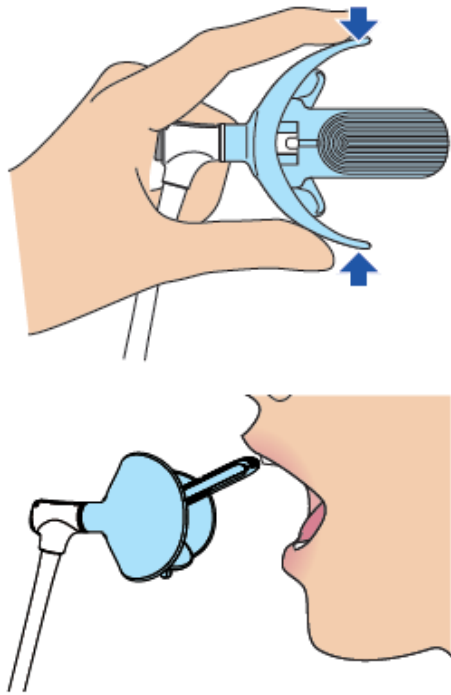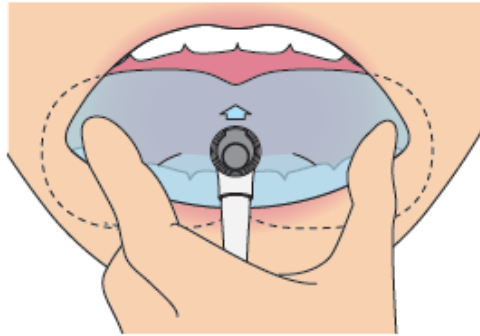
Insert the oral interface in your mouth by squeezing the lip shield appropriately.

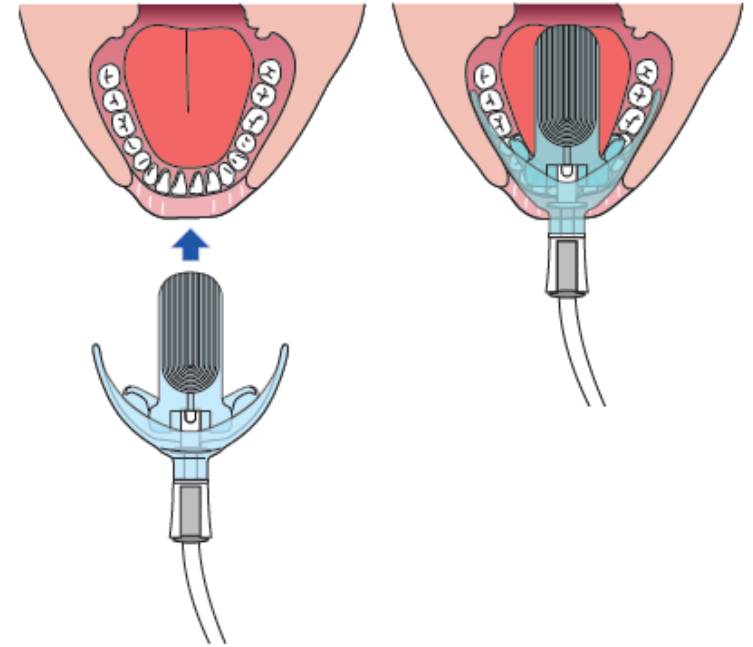Make sure the lip shield is located between the teeth and the lips.

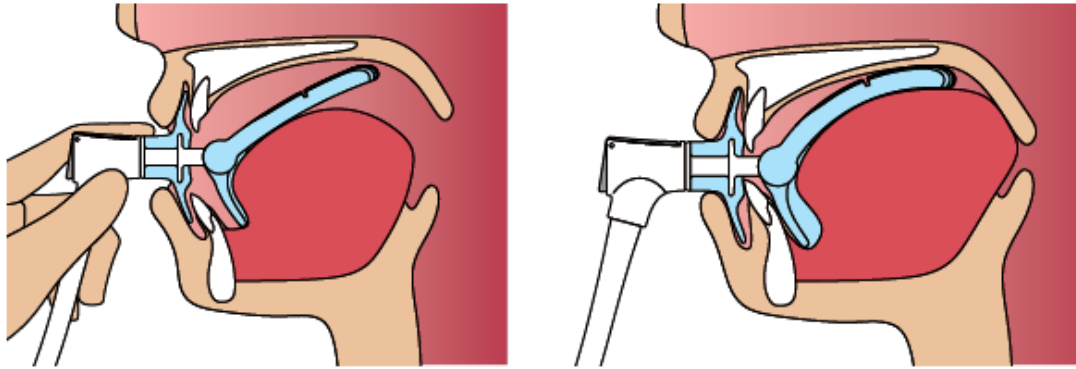Keep the lip shield and the flexible plate in your mouth, then rest the tongue on the tongue shield.


Close your mouth and get ready to start the treatment .


**Note** Please put the oral interface ONLY in your mouth.

**Note** DO NOT chew on or swallow the oral interface.

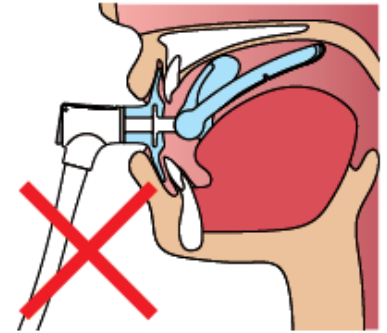**Note** DO NOT wear the oral interface while eating or drinking.

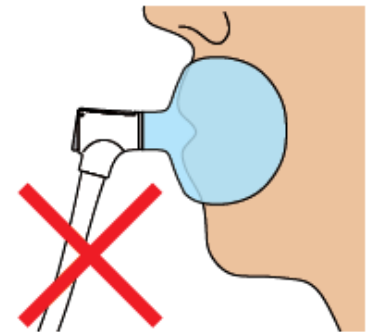**!** **Caution** DO NOT put tongue shield on the tongue.

**!** **Caution** DO NOT put lip shield outside the lip.

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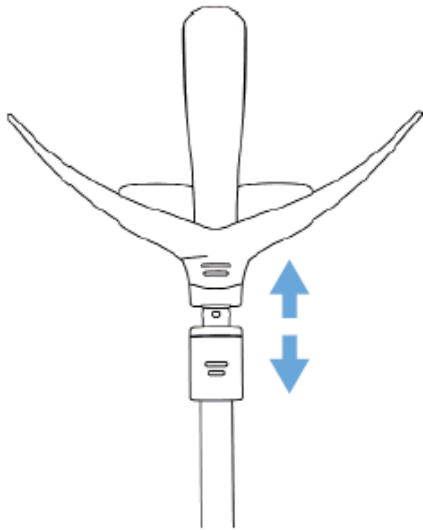
- Disconnect the oral interface from the oral interface connector.


I07/I07M/I07M1/I07MT/I07S

- Clean the used oral interface


Rinse and gently rub the oral interface with cold water.

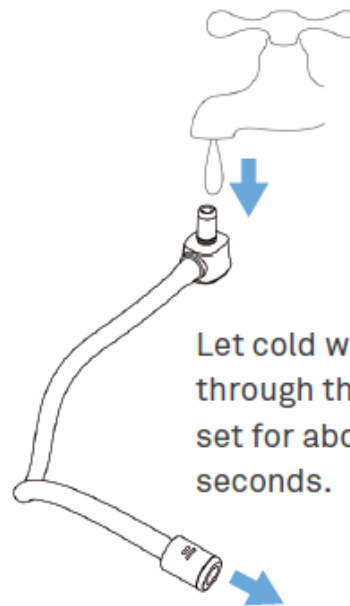
**Note** You can use a soft brush or a sponge to gently scrub the surface of the oral interface.

Let cold water flow through the tubing set for about 30 seconds.

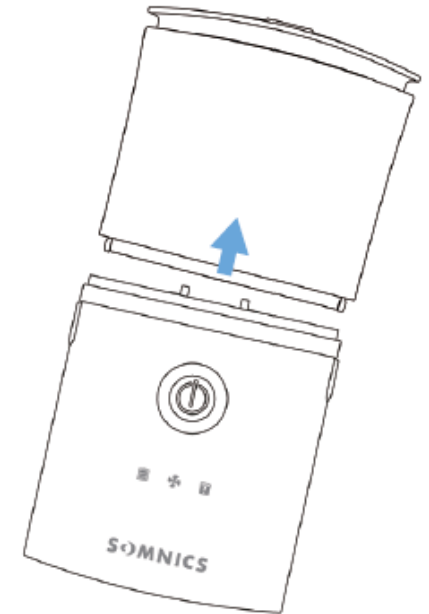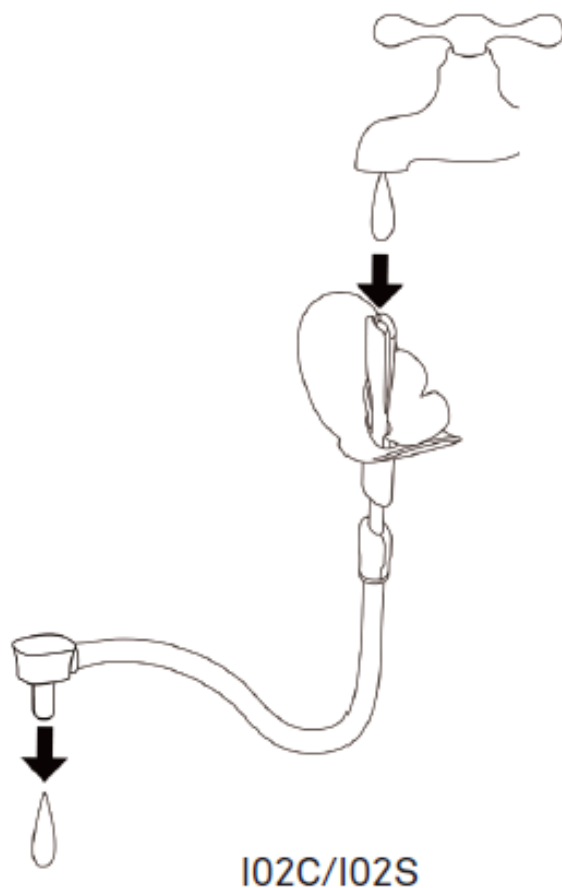- Disconnect the console and the container

Rinse and gently rub the oral interface with cold water.

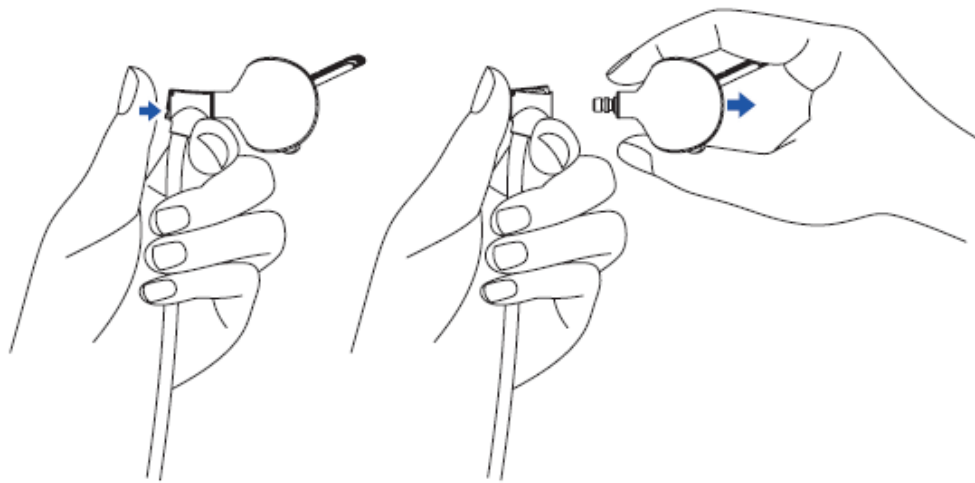
Let cold water flow through the tubing set for about 30 seconds.

**Note** You can use a soft brush or a sponge to gently scrub the surface of the oral interface.

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
Unlock the oral interface from the oral interface pressing the latch on joint lock then pull.

**Note** You can use a soft brush or a sponge to gently scrub the surface of the oral interface.

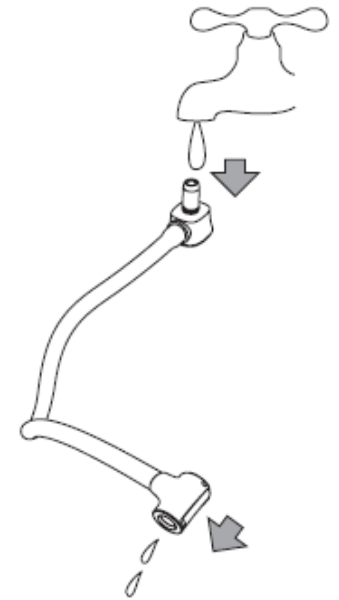Rinse and gently rub the oral interface with cold water.


Let cold water flow through the tubing set for about 30 seconds.

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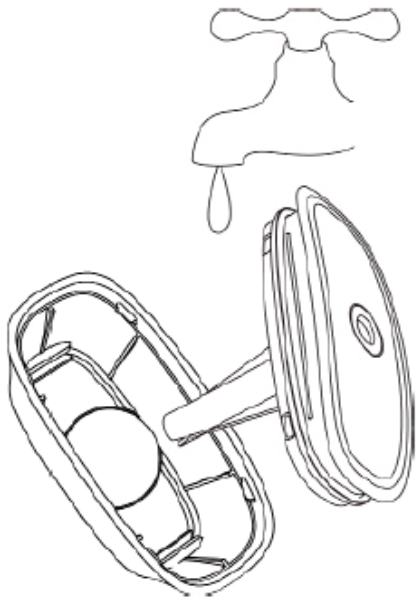• Clean the used container

**Note** If the container body is not wetted by saliva, you can use a soft brush or a sponge to gently scrub the surface of the container.

**Note** Rinse the container with cold water. DO NOT touch or brush the filter inside the container body.

**Note** Shake off the excess water, and then dry the container, the tubing set and the oral interface with a clean towel.

**⚠ Caution** DO NOT wash, touch or brush the filter on the bottom of the container body. Dry the filter when it has been wetted by saliva or water.

**Note** DO NOT clean the oral interface in the dishwasher.

**Note** DO NOT clean the oral interface with toothpaste.

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

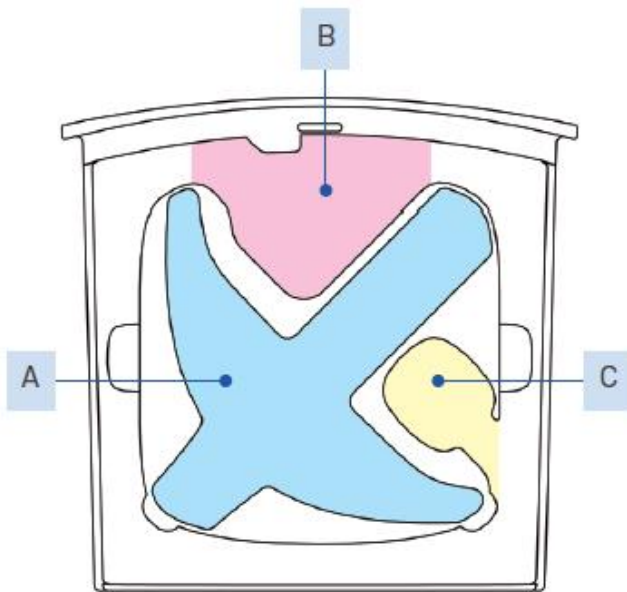
Put the oral interface in the region A.

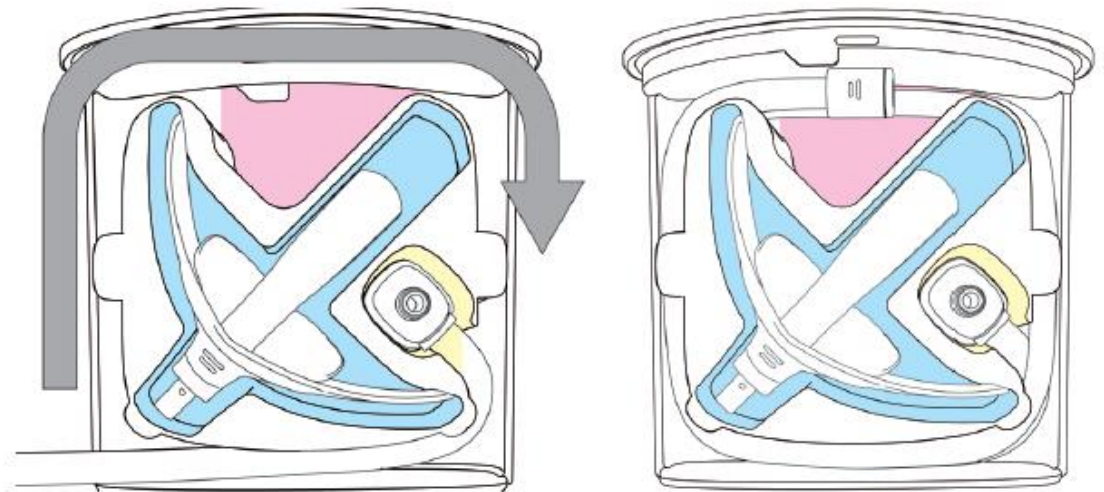Put the device connector in the region C and circle the tubing set in clockwise direction. Eventually, rest the oral interface connector in the region B.

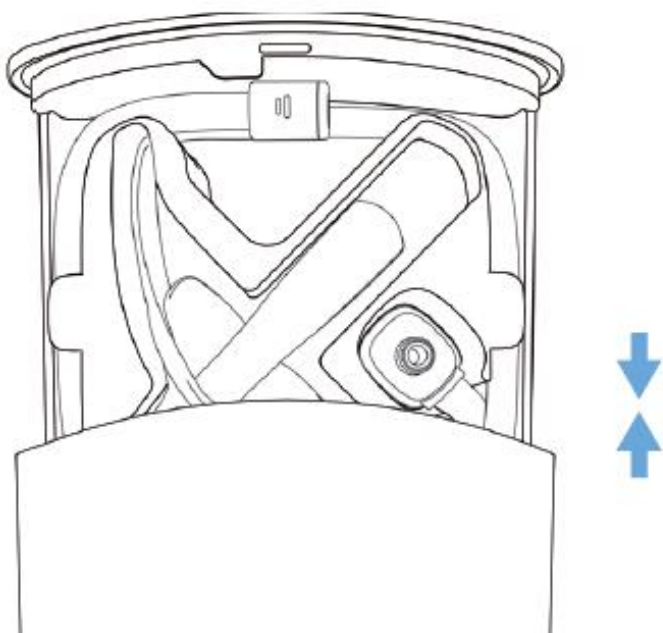
Combine the two parts of oral interface box.

**Note** Please make sure the tubing will not get tangled or kinked when storage.

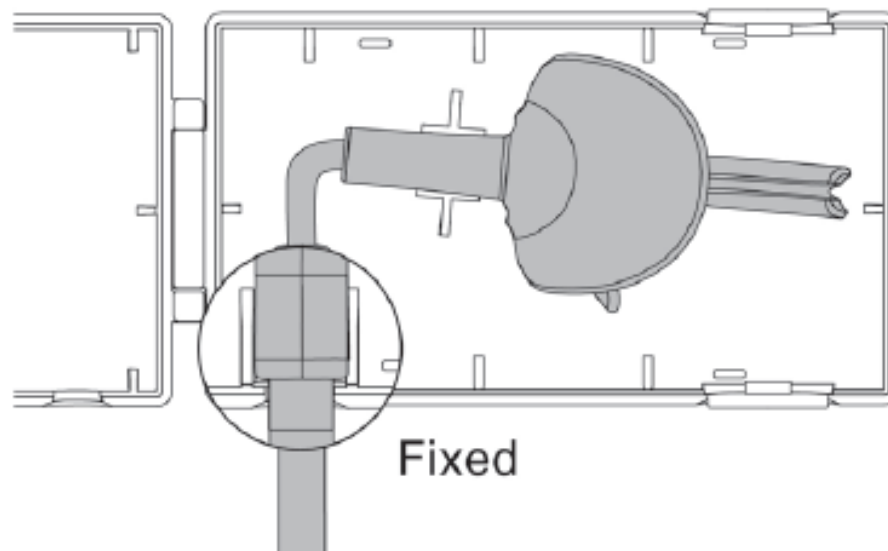B. Oral interface box for I02C/I02S.

Put the oral interface in the oral interface box.

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Problem	Possible Cause	Light Status	Solution
---------	----------------	--------------	----------

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Product Name	iNAP One Sleep Therapy System
Model Name of iNAP One console	R07-A (A=Adjustable), R07-V (V= Valve)
Model Name of Container	C07
Model Name of Oral Interface	I07/I07M/I07M1/I07MT/I07S/I02C/I02S/I08
Model Name of Tubing Set	T01/T03(except for I02C/I02S )
Model Name of DryPad	A01/A03
Model Name of Power Adapter	P01/P03
Dimensions (Console and Container)	5.98" x 3.14" x 1.41"(152 x 80 x 36mm)
Weight (Console and Container)	0.47 lbs (210g)
Warranty/Life time (Console)	2 years
Battery Run Life	Approximately 4 nights (8 hours per night)

<b>Product Name</b>	<b>iNAP One Sleep Therapy System</b>
Bluetooth Module	Frequency Range: 2.402 GHz - 2.480 GHz Transmit Power: Class 2 output power (0 dBm typical, +2 dBm maximum)
Power Adapter	Input: 100-240 Vac, 50~60Hz Output: 5.0Vdc, 0~2.0A (ATM012-W050E/U/B, Adapter Technology Co., Ltd. )
	Input: 100-240 Vac, 50~60Hz Output: 5.0Vdc, 0~2.4A (HPU15-102, Sinpro Electronics Co., Ltd.)
Storage/Transport	-20°C ~ 50°C
	15 ~ 93% (non condensing)
Operating Range	5°C ~ 40°C
	15 ~ 93% (non condensing)
Operation/Storage Atmospheric pressure:	700hPa ~ 1060hPa
Operating Altitude	Up to 8,858ft (2700m)

<b>Product Name</b>	<b>iNAP One Sleep Therapy System</b>
Vacuum Setting / Accuracy	-20~-90mmHg / ± 5mmHg *
Ingress Protection	IP22
Patient Protection	Type BF
Compliance	IEC 60601-1    IEC 60601-1-2    IEC 60601-1-11

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Manufacturer's declaration-electromagnetic emissions		
The iNAP One is intended for use in the electromagnetic environment (for home healthcare) specified below. The customer or the user of the iNAP One should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment-guidance (for home healthcare environment)
RF emissions CISPR 11	Group 1	The iNAP One uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference with nearby electronic equipment.
RF emissions CISPR 11	Class B	The iNAP One is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Not applicable	

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance (for home healthcare environment)
Electrostatic discharge (ESD) IEC 61000-4-2	Contact:±8 kV Air±2 kV, ±4 kV, ±8 kV, ±15 kV	Contact:±8 kV Air±2 kV, ±4 kV, ±8 kV, ±15 kV	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrical fast transient/ burst IEC 61000-4-4	+ 2kV for power supply lines + 1kV for input/ output lines	Not applicable  Not applicable	Main power quality should be that of a typical home healthcare environment.
Surge IEC 61000-4-5	+ 0.5kV, +1kV line(s) to line(s) + 0.5kV, +1kV,+ 2kV line(s) to earth	Not applicable  Not applicable	Mains power quality should be that of a typical home healthcare environment.
Voltage Dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Voltage dips: 0 % $U_T$ ; 0,5 cycle 0 % $U_T$ ; 1 cycle 70 % $U_T$ ; 25/30 cycles  Voltage interruptions: 0 % $U_T$ ; 250/300 cycle	Voltage dips: Not applicable Not applicable Not applicable  Voltage interruptions: Not applicable	Main power quality should be that of a typical home healthcare environment. If the user of the iNAP One requires continued operation during power interruptions, it is recommended that the iNAP One be powered from an uninterruptible power supply or a battery.
Power frequency(50, 60 Hz) magnetic field IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz and 60 Hz	The iNAP One power frequency magnetic fields should be at levels characteristic of a typical location in a typical home healthcare environment.

NOTE  $U_T$  is the a.c. mains voltage prior to application of the test level.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms: 0,15 MHz – 80 MHz 6 Vrms: in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	3 Vrms: 0,15 MHz – 80 MHz 6 Vrms: in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	Portable and mobile RF communications equipment should be used no closer to any part of the iNAP One including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	<p><b>Recommended separation distance:</b>  <math>d = 1,2 \sqrt{P}</math>  <math>d = 1,2 \sqrt{P}</math> 80MHz to 800 MHz  <math>d = 2,3 \sqrt{P}</math> 800MHz to 2,7 GHz</p> <p>Where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in meter (m).</p>
Radiated RF IEC 61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

## Recommended separation distance between portable and mobile RF communications equipment and the iNAP One

The iNAP One is intended for use in an electromagnetic environment (for home healthcare) in which radiated RF disturbances are controlled. The customer or the user of the iNAP One can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the iNAP One as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	N/A	0,12	0,23
0,1	N/A	0,38	0,73
1	N/A	1,2	2,3
10	N/A	3,8	7,3
100	N/A	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $p$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

## Manufacturer's declaration-electromagnetic immunity

### Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment

The iNAP One is intended for use in the electromagnetic environment (for home healthcare) specified below. The customer or the user of the iNAP One should assure that it is used in such an environment.

Test frequency (MHz)	Band <sup>a)</sup> (MHz)	Service <sup>a)</sup>	Modulation <sup>b)</sup>	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)	Compliance LEVEL (V/m) (for home healthcare)
385	380 – 390	TETRA 400	Pulse modulation b) 18 Hz	1,8	0,3	27	27
450	430 – 470	GMRS 460, FRS 460	FM c) ±5 kHz deviation 1 kHz sine	2	0,3	28	28
710 745 780	704 – 787	LTE Band 13, 17	Pulse modulation b) 217 Hz	0,2	0,3	9	9
810 870 930	800 – 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation b) 18 Hz	2	0,3	28	28
1 720 1 845 1 970	1 700 – 1 990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation b) 217 Hz	2	0,3	28	28

2 450	2 400 – 2 570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation b) 217 Hz	2	0,3	28	28
5 240 5 500 5 785	5 100 – 5 800	WLAN 802.11 a/n	Pulse modulation b) 217 Hz	0,2	0,3	9	9

NOTE If necessary to achieve the IMMUNITY TEST LEVEL, the distance between the transmitting antenna and the ME EQUIPMENT or ME SYSTEM may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

a) For some services, only the uplink frequencies are included.

b) The carrier shall be modulated using a 50 % duty cycle square wave signal.

c) As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

Design Life	
Device, power supply unit:	2 years
Air tubing, connectors	3 months
Saliva container	6 months
DryPad	Dispose per use
General	The patient is an intended operator

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Warning or caution

Refer to instruction manual

Product manufacturer

Date of manufacture

Type BF, applied part

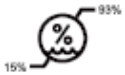
Serial number


Batch number

Storage/Transport temperature limits

Storage/Transport humidity Limits

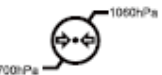Operation/Storage atmospheric pressure limits

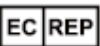
Prescription only medical device

Protection against ingress of solids larger than 12.5 mm and drip water

The device conforms to the Medical Devices Directive (93/42/EEC)

Symbol for "AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY"

Do not use if package is damaged

Keep dry

The symbol of the crossed out wheeled bin indicates that the product (electrical and electronic equipment) should not be placed in municipal waste. Please check local regulations for disposal of electronic products.

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